# GOOD LABORATORY PRACTICE

# SAMPLE COLLECTION - URINE-CLEAN VOIDED

# BEFORE COLLECTION

- ♦ Train on procedure
- ♦ Check requisition and patient ID
- ♦ Get supplies
  - Sterile container patient cleansing kit and patient instructions
- Specimen required
- Freshly voided urine (first morning specimen preferred) Fresh bladder catheter specimen or suprapubic aspiration are acceptable
- ♦ Label container
  - Patient name, unique identifier, date and time collected
- ♦ Wash your hands put on gloves

## INSTRUCT PATIENT

- Instruct patient in cleansing and obtaining midstream sample
- Provide written instructions to the patient
- ♦ Verify the instructions with patient

### AFTER COLLECTION

- ♦ Put on gloves to handle sample
- ♦ Screw lid on tightly
- ♦ Put specimen in a biohazard bag for transport
- Deliver fresh samples to testing laboratory immediately or refrigerate samples at 2 - 8°C or transfer to a preservative tube if not cultured immediately



This publication was supported 100% by Cooperative Agreement Number #U60/CD303019 from Centers for Disease Control and Prevention ("CDC"). Its contents are solely the responsibility of the authors and do not necessarily represent the official views of CDC.

